

HemoTemp® II Blood Bag Temperature Indicator

Easy to Use, Accurate, and Reliable

1) Activate - Place in the Activator for 60 seconds. Remove from activator and rest until the flower begins to turn blue - then it is ready to be placed on the bag.



2) Adhere to Blood Bag - Remove backing liner from the HemoTemp II and press the indicator onto the center or near the bottom of the refrigerated blood bag.



3) Easy to Read - The Reversible windows show the current temperature of the blood bag.

4) The irreversible flower window will turn tan/brown once the blood bag has exceeded 10C, regardless of the current temperature of the blood bag.

